


Exclusively from **Vetsonic**

GYN-stick the ultimate Obstetric Aid

Correction of Displaced Limbs



Turn the fine rope with the metal spoon around the proximal end of the displaced leg. Close the circle extra-vaginally on the end of the GYN-Stick; direct this loop in distal direction of the leg in order to get closer to the assistant and for further actions assisting the calving process.

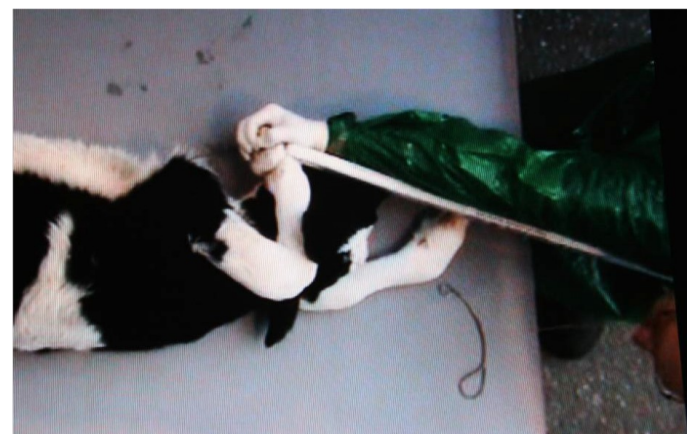

Vetsonic (UK) Ltd

Riccal Drive · York Road Business Park · Malton · YO17 6YE · UK


Detorsion with One Rope/One Person



Detorsion with Two Ropes/Two Persons



Attach the ropes and fix them to the GYN-stick extra-vaginally. Tighten the GYN-stick which will automatically bring the legs together. Fix the ropes outside on the distal hole of the GYN-stick, in preparation for turning.

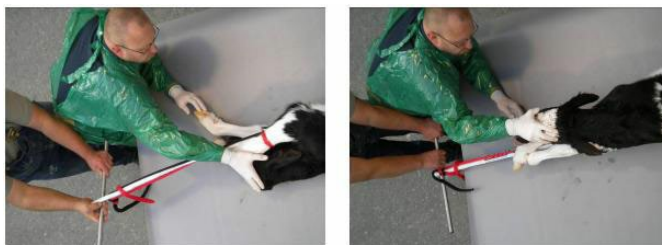

Insert the metal bar while the vet is monitoring the position of the calf within the cow and simultaneously turn the wheel. The smooth shape of the GYN-stick will reduce the incidence of uterine rupture and/or damage to the calf.

Vetsonic (UK) Ltd

Riccal Drive · York Road Business Park · Malton · YO17 6YE · UK